Instructions for use

myJunior kit

sectional
matrix
systems

en - ENGLISH	2	da - DANSK	42
de - DEUTSCH	6	cs - ČEŠTINA	46
fr - FRANÇAIS	10	tr - TÜRKÇE	50
it - ITALIANO	14	el - ΕΛΛΗΝΙΚΑ	54
es - ESPAÑOL	18	bg - БЪЛГАРСКИ	58
pt - PORTUGUÊS	22	hu - MAGYAR	62
no - NORSK	26	lv - LATVIEŠU VALODA	66
et - EESTI KEEL	30	lt - LIETUVIŲ KALBA	70
fi - SUOMI	34	ro - LIMBA ROMÂNĂ	74
sv - SVENSKA	38	sk - SLOVENČINA	78

Medical Devices Kit for professional use only
All individual components are CE marked
Sectional matrix systems

Instructions for use

Description

myJunior kit: sectional matrix system for the restoration of posterior deciduous and young permanent teeth.

Composition

myClip Junior, myRing Junior: NiTi Ring with thermoplastic overmolding | myTines Junior: thermoplastic | Silicone Tubes: silicone
LumiContrast matrices: stainless steel | Quickmat matrices: stainless steel | Wood wedge: maple wood | myWedge: thermoplastic.

Indications

Restoration of posterior deciduous and young permanent teeth.

Contraindications

myJunior kit is not intended to be used in the case of uncooperative young patients.

Side effects

There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydentia.

Interactions with other medical devices or products

There are no known interactions.

Information for the patient

No information for the patient related to the myJunior kit is necessary.

Shelf life

Please refer to the expiration date on the packaging. Expiration date refers to wooden wedges and silicone tubes.

Preservation and storage


Store in a clean, dry environment away from moisture or excessive humidity.

Warnings

Due to the presence of small and sharp components, the use of a rubber dam is mandatory to avoid accidental ingestion or aspiration. Always ensure the rings using dental floss. In case of accidental inspiration of a myJunior kit component (matrix, retaining ring and its extremities, wedge) seek immediate medical attention. In case of ingestion of a myJunior kit component, it is strongly recommended seeking for medical assistance and locate the ingested components. If illness develops after ingestion, consult a doctor immediately. All Polydentia reusable Products are supplied non-sterile and must be appropriately disinfected, cleaned and sterilized before the first use. Please refer to the Cleaning, disinfection and Sterilization Instructions at www.polydentia.ch.

Application

① **Select the appropriate ring:** Select the appropriate ring and extremities according to the clinical situation. ② **Prepare the cavity:** Remove the carious lesion and prepare the tooth with a conservative Class II preparation. Even if the contact was not totally broken the matrix can be placed. ③ **Place the matrix:** Select an appropriate matrix band according to the tooth morphology and restoration type. Place the contoured sectional matrix with the concave edge towards the occlusal margin using your fingers or with tweezers. ④ **Wedging:** Select the appropriate sized plastic or wooden wedge to close the gingival margin, and insert it firmly. When needed maintain a finger on the matrix band to avoid dislodging during wedging. ⑤ **Place the ring:** Open the ring (myClip Junior by using your fingers / myRing Junior by ring forceps). Enlarge it just enough to position it on the tooth. Place the ring carefully as far gingivally as possible. ⑥ **Burnish the matrix:** For an excellent contact burnish the band against the adjacent tooth, using a ball burnisher or tweezer. ⑦ **Restore the cavity:** Restore the cavity according to the filling material manufacturer's instructions. ⑧ **Remove the sectional matrix system:** Remove the ring, the wedge and the matrix. Our suggestion for an easy matrix removal is to use the special Polydentia myQuickmatrix Forceps. ⑨ **Finishing and polishing:** With thin burs remove any small excess of material then polish the restoration according to manufacturer's recommendations. **Use only with rubber dam.**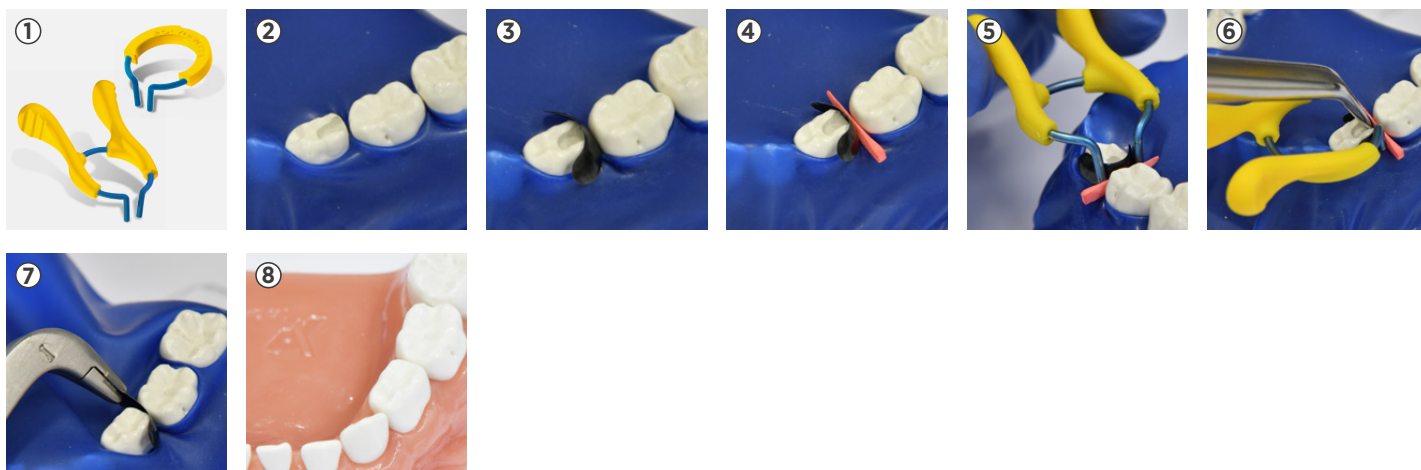

Disposal

Because of their composition, matrices and wedges can not be sterilized and are therefore single use devices. Do not reuse them. Properly dispose devices contaminated with potentially infectious substances of human origin, in accordance with local regulations.

Ring reconditioning


For a proper, effective and safe reconditioning of myRing Junior, myClip Junior and their extremities read the Cleaning, Disinfection and Sterilization Instructions at www.polydentia.ch.

Select and customise your sectional matrix ring

Select a myJunior ring


Ⓐ **myClip Junior:** myClip Junior is an optimal solution for easy and quick placement. This ring does not need forceps, in fact it can be placed directly with one's own fingers.

Ⓑ **myRing Junior:** myRing Junior is the smallest Polydentia sectional matrix ring, and therefore the optimal solution for the youngest patients. As all standard sectional matrix rings, it needs a pair of forceps for the placement. For an easy placement of the myRing Junior we recommend to use our myForceps.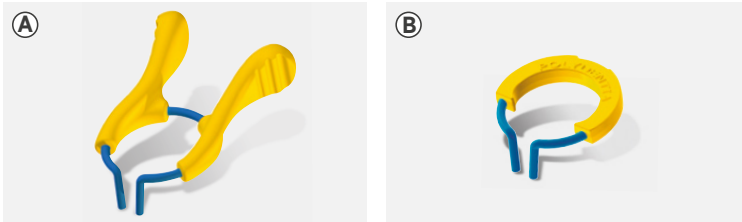

Customise your sectional matrix ring


Ⓒ **Silicone Tubes:** Use the rings myJunior with Silicone Tubes in case of wider class II cavities, where an increased adaptation of the matrix is necessary or when restoring two adjacent cavities. Break the wood wedge for an improved stability of the ring. Ⓓ To accommodate the ring with silicone tubes break the wood wedge. Ⓔ **myTines Junior:** myClip and myRing Junior in combination with myTines Junior provide outstanding stability of the ring even in case of very young and small primary teeth. How to position myTines Junior: **1.** insert the extremities into the ring ends with the orientation index facing buccally and palatally (as shown in the image). **2.** turn the extremities so that the forked gaps align with the wedge. Firmly hook myTines Junior on the wedge once positioned. Ⓕ **No extremities:** The rings myJunior can be used without extremities in case of small class II cavities.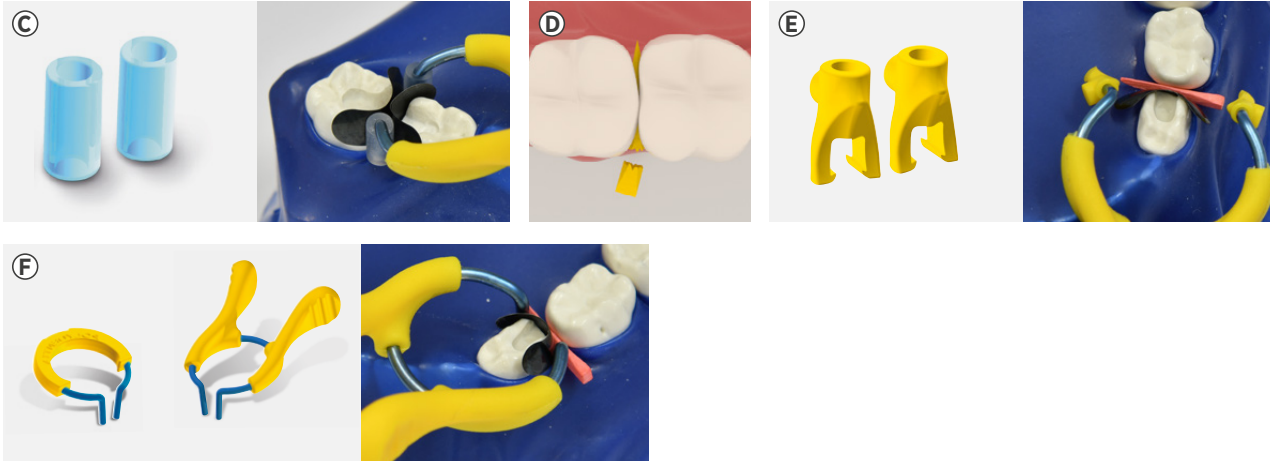

How to remove myClip Junior & myRing Junior with myTines Junior


Removal of myJunior rings from a PLASTIC myWedge

Ⓒ myTines Junior are developed to firmly hook on the wedge. Incorrect removal might cause the detachment of myTines from myClip Junior/myRing Junior. Ⓓ **1.** Open the rings myJunior and let the lingual/palatal ring extremity myTines Junior slide out from the plastic wedge. **2.** Rotate the ring to unhook the buccal ring extremity myTines Junior.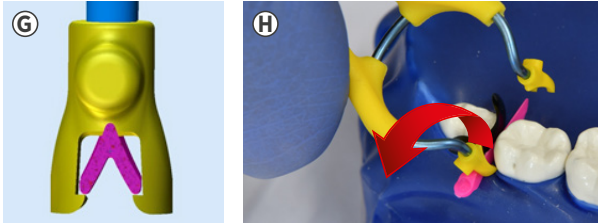

Removal of myJunior rings from a WOODEN wedge

Ⓘ myTines Junior are developed to firmly hook on the wedge. Incorrect removal might cause the detachment of myTines from myClip Junior/myRing Junior. Ⓛ Open the rings myJunior and let both lingual/palatal and buccal ring extremities myTines Junior slide out from the wooden wedge.

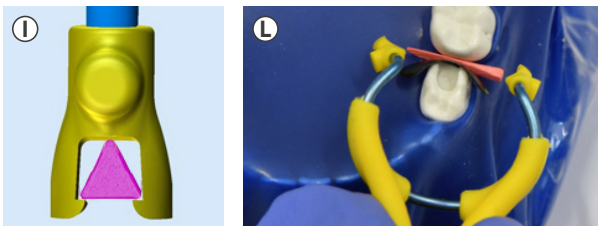Select an appropriate sectional matrix

Young, small primary molars

Junior LumiContrast Matrices: Height: 3.75 mm, Thickness: 0.04 mm. Junior Quickmat Matrices: Height: 3.75 mm, Thickness: 0.04 mm.

Primary molars, Young permanent premolars

LumiContrast Matrices: Height: 5 mm, Thickness: 0.04 mm. Quickmat Matrices: Height: 5 mm, Thickness: 0.04 mm.

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© myTines Junior sind so entwickelt, dass sie fest am Keil einhaken. Ein unsachgemäßes Entfernen kann dazu führen, dass sich die myTines vom myClip Junior/myRing Junior ablösen. H 1. Öffnen Sie die myJunior Ringe und lassen Sie das linguale/palatinale Ringende myTines Junior aus dem Kunststoffkeil gleiten. 2. Drehen Sie den Ring, um das bukkale Ringende myTines Junior auszuhaken.