Instructions for use

myQuickmat classico kit

sectional
matrix
systems

 polydentia
swiss manufacture 

en - ENGLISH	2
de - DEUTSCH	5
fr - FRANÇAIS	8
it - ITALIANO	11
es - ESPAÑOL	14
pt - PORTUGUÊS	17
no - NORSK	20
et - EESTI KEEL	23
fi - SUOMI	26
sv - SVENSKA	29

da - DANSK	32
cs - ČEŠTINA	35
tr - TÜRKÇE	38
el - ΕΛΛΗΝΙΚΑ	41
bg - БЪЛГАРСКИ	44
hu - MAGYAR	47
lv - LATVIEŠU VALODA	50
lt - LIETUVIŲ KALBA	53
ro - LIMBA ROMÂNĂ	56
sk - SLOVENČINA	59

Medical Devices Kit for professional use only
All individual components are CE marked
Sectional matrix systems

Instructions for use

Description

myQuickmat Classico kit: Complete sectional matrix system developed to offer gentle tooth separation in order to create contact points in the restoration of class II cavities. This kit features Polydentia's myRing Classico, a classic sectional matrix ring made of nickel titanium. It is reinforced to provide an optimal and gentle separation force in combination with the included Interdental Maple Wood Wedges. For the restoration of very extensive cavities, the kit also includes the Delta silicone tubes and is compatible with the Diamond24 silicone tubes (not included). These anatomically shaped silicone tubes ensure optimal matrix retention and adaptation to the tooth's anatomy. LumiContrast sectional matrices complete the kit's assortment. These dark-blue matrices significantly increase contrast and reduce glare effect, especially while using loupes or microscopes.

Composition

myRing Classico: NiTi Ring with thermoplastic overmolding | Delta Tubes: silicone | LumiContrast matrices: Stainless steel
Quickmat matrices: Stainless steel | Wood wedge: Maple wood

Indications

myQuickmat Classico kit is indicated for gentle tooth separation in order to create correct contact points in the restoration of class II cavities.

Contraindications

None known.

Side effects

There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydentia.

Interactions with other medical devices or products

There are no known interactions.

Information for the patient

No information for the patient related to myQuickmat Classico kit is necessary.

Shelf life

Please refer to the expiration date on the packaging. Expiration date refers to wooden wedges and silicone tubes.

Preservation and storage


Store in a clean, dry environment away from moisture or excessive humidity.

Warnings

Due to the presence of small and sharp components, the use of a rubber dam is mandatory to avoid accidental ingestion or aspiration. Always ensure the rings using dental floss. In case of accidental inspiration of a myQuickmat Classico kit component (matrix, retaining ring and its extremities, wedge) seek immediate medical attention. In case of ingestion of a myQuickmat Classico Kit component, it is strongly recommended seeking for medical assistance and locate the ingested components. If illness develops after ingestion, consult a doctor immediately. All Polydentia reusable Products are supplied non-sterile and must be appropriately disinfected, cleaned and sterilized before the first use. Please refer to the Cleaning, Disinfection and Sterilization Instructions at www.polydentia.ch.

Application

① **Prepare the cavity.** Remove the carious lesion and prepare the tooth with a conservative Class II preparation. Even if the contact was not totally broken the matrix can be placed. ② **Place the matrix.** Select an appropriate matrix band according to the tooth morphology and restoration type. Place the contoured sectional matrix with the concave edge towards the occlusal margin using your fingers or with tweezers. ③ **Wedging.** Select the appropriate sized wooden wedge to close the gingival margin, and insert it firmly. When needed maintain a finger on the matrix band to avoid dislodging during wedging. ④ **Place myRing Classico.** Expand myRing Classico using Polydentia's myForceps or standard rubber dam forceps. Place the ring carefully and securely. Please refer to the Tips & Tricks section to see the different ways to position myRing Classico and ensure best teeth separation as well as matrix adaptation. ⑤ **Burnish the matrix.** For an excellent contact burnish the band against the adjacent tooth, using a ball burnisher or tweezers. ⑥ **Restore the cavity.** Restore the cavity according to the filling material manufacturer's instructions. ⑦ **Remove the sectional matrix system.** Remove myRing Classico, the wedge and the matrix. Our suggestion for an easy matrix removal is to use the special Polydentia myQuickmatrix Forceps. ⑧ **Finishing and polishing.** With thin burs remove any small excess of material then polish the restoration according to manufacturer's recommendations.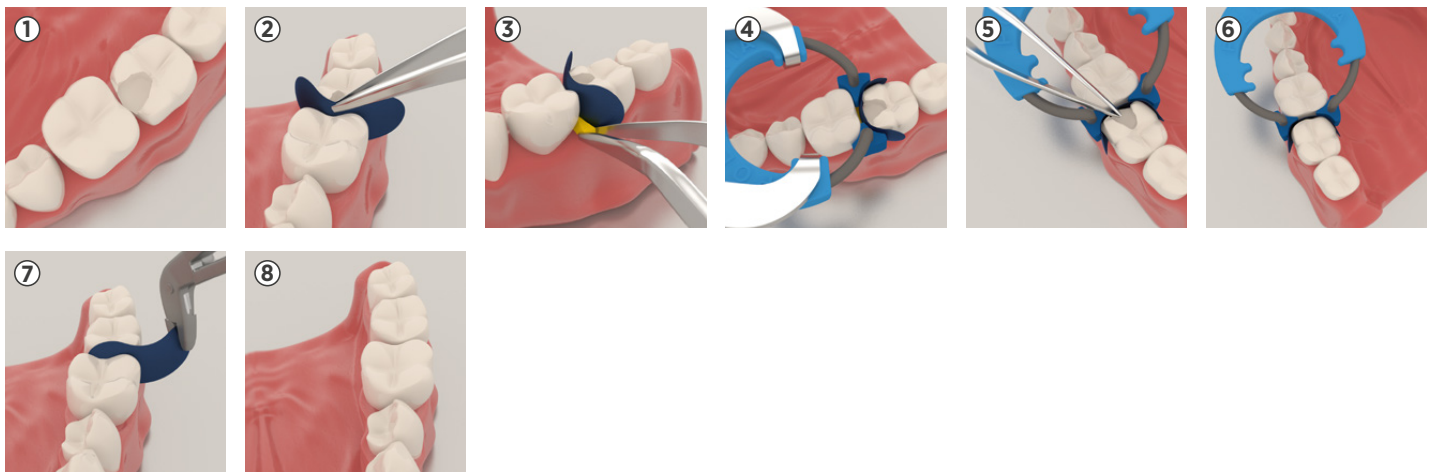

Disposal

Because of their composition, matrices can not be sterilized and are therefore single use devices. Do not reuse them. Properly dispose devices contaminated with potentially infectious substances of human origin, in accordance with local regulations.


Ring reconditioning

For a proper, effective and safe reconditioning of the myRing Classico and their extremities read the Cleaning, Disinfection and Sterilization Instructions at www.polydentia.ch.

Tips & tricks using myQuickmat Classico kit

Wedging ^(A)

According to the clinical situation, the wooden wedge can be broken in order to better accommodate myRing Classico with its extremities. This trick is very helpful in case of small teeth. ❌ NO break. ✅ Break.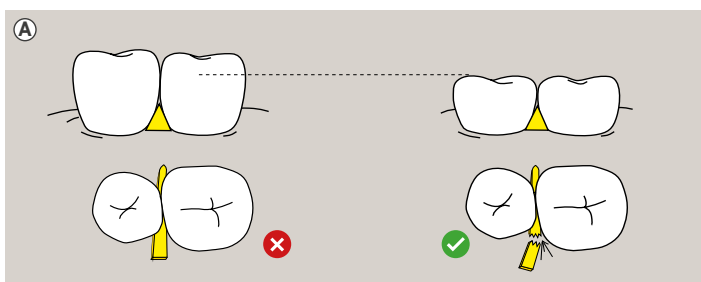


Delta Tubes placement options

Delta Tubes can be oriented according to the clinical situation to obtain the best matrix retention and adaptation to the tooth's anatomy. The most common cases are explained as follows (Case A, B). Different orientation possibilities can be combined to best adapt myRing Classico to each specific clinical situation. The use of Delta Tubes is not mandatory, myRing Classico can be placed also without Delta Tubes (Case C, D).


Case A, small cavities [ⓑ]

Delta Tubes edges allow a perfect adaptation between the teeth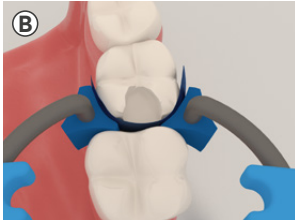

Case B, broad cavities [ⓒ]

Delta Tubes flat surfaces provide support on the adjacent tooth, no matrix deformation.

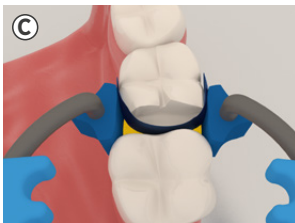Case C and D, myRing Classico without Delta Tubes [ⓓ] [ⓔ]

myRing Classico tines placed near to the wood wedge: This placement technique is suitable for cases of a wide space in the interproximal area.
myRing Classico tines placed over the wood wedge: This placement technique is suitable if the interproximal space is tight or when the cavity is deep.

MOD and multiple tooth restorations [ⓕ]

myQuickmat Classico rings are designed to be stacked over one another to perform a MOD and multiple tooth restorations. **1.** Prepare the cavities as described in the instructions for use, step 1. **2.** Place matrices and wedges on both mesial and distal surfaces of the tooth being restored. **3.** Position myRing Classico. **4.** Complete the restoration as described in the instructions for use, steps 5 to 8.

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