Instructions for use

myQuickmat Forte starter kit

sectional
matrix
systems

en - ENGLISH	2	da - DANSK	32
de - DEUTSCH	5	cs - ČEŠTINA	35
fr - FRANÇAIS	8	tr - TÜRKÇE	38
it - ITALIANO	11	el - ΕΛΛΗΝΙΚΑ	41
es - ESPAÑOL	14	bg - БЪЛГАРСКИ	44
pt - PORTUGUÊS	17	hu - MAGYAR	47
no - NORSK	20	lv - LATVIEŠU VALODA	50
et - EESTI KEEL	23	lt - LIETUVIŲ KALBA	53
fi - SUOMI	26	ro - LIMBA ROMÂNĂ	56
sv - SVENSKA	29	sk - SLOVENČINA	59

Medical Devices Kit for professional use only
All individual components are CE marked
Sectional matrix systems

Instructions for use

Description

This kit features myRing Forte, a sectional matrix ring that adapts to both teeth of different crown heights and wide cavity preparations, thanks to its extremities (myTines Small, myTines Medium and myTines Large). In case of dissimilar tooth morphologies, the autoclavable and replaceable myTines can be used interchangeably to enhance matrix adaptation. The kit comprises a selection of myWedge, high-quality, anatomical interdental plastic wedges. Their hollow V-shape allows them to compress and adapt to the interproximal anatomy of the teeth, preventing damage to the papilla. LumiContrast and QuickmatFLEX sectional matrices complete the kit's matrix assortment.

Composition

myRing Forte: Stainless steel | myTines: thermoplastic | LumiContrast matrices: Stainless steel
QuickmatFLEX matrices: Titanium | myWedge: thermoplastic

Indications

myQuickmat Forte starter kit is indicated for strong tooth separation in order to create contact points in the restoration of Class II cavities. myRing Forte's anatomically shaped reusable tines ensure perfect matrix adaptation and myWedge forms the matrix band to the cervical area of the tooth.

Contraindications

None known.

Side effects

There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydentia.

Interactions with other medical devices or products

There are no known interactions.

Information for the patient

No information for the patient related to myQuickmat Forte starter kit is necessary.

Preservation and storage


Store in a clean, dry environment away from moisture or excessive humidity.

Warnings

Due to the presence of small and sharp components, the use of a rubber dam is mandatory to avoid accidental ingestion or aspiration. Always ensure myRing Forte using dental floss. In case of accidental inspiration of a myQuickmat Forte starter kit component (matrix, ring and its extremities, wedge) seek immediate medical attention. In case of ingestion of a myQuickmat Forte starter kit component, it is strongly recommended seeking for medical assistance and locate the ingested components. If illness develops after ingestion, consult a doctor immediately. All Polydentia reusable Products are supplied non-sterile and must be appropriately disinfected, cleaned and sterilized before the first use. Please refer to the Cleaning, disinfection and Sterilization Instructions at www.polydentia.ch.

Application

① **Prepare the cavity:** Remove the carious lesion and prepare the tooth with a conservative Class II preparation. Even if the contact was not totally broken the matrix can be placed. ② **Place the matrix:** Select an appropriate matrix band according to the tooth morphology and restoration type. Place the contoured sectional matrix with the concave edge towards the occlusal margin using your fingers or with tweezers. ③ **Wedging:** Select the appropriate sized myWedge to close the gingival margin, and insert it firmly. When needed maintain a finger on the matrix band to avoid dislodging during wedging. ④ **Place myRing Forte:** Expand myRing Forte using Polydentia's myForceps; enlarge it just enough to position it on the tooth. Carefully place myRing Forte over the wedge as far gingivally as possible. ⑤ **Burnish the matrix:** For an excellent contact burnish the band against the adjacent tooth, using a ball burnisher or tweezers. ⑥ **Restore the cavity:** Restore the cavity according to the filling material manufacturer's instructions. ⑦ **Remove the sectional matrix system:** Remove myRing Forte, the wedge and the matrix. Our suggestion for an easy matrix removal is to use the special Polydentia myQuickmatrix Forceps. ⑧ **Finishing and polishing:** With thin burs remove any small excess of material then polish the restoration according to manufacturer's recommendations.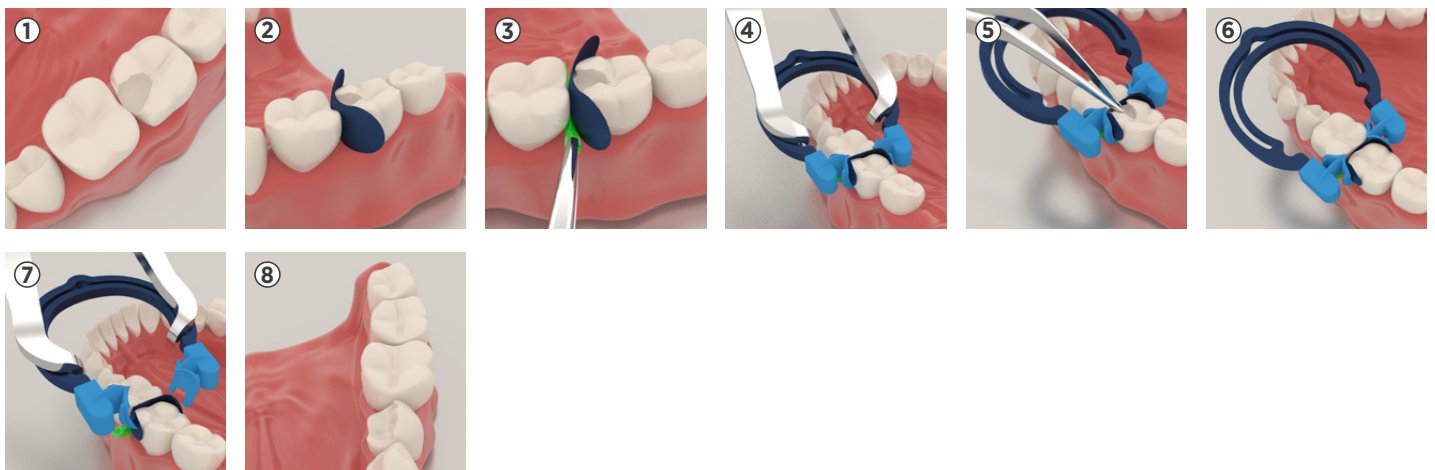

Disposal

Because of their composition, matrices and wedges can not be sterilized and are therefore single use devices. Do not reuse them. Properly dispose devices contaminated with potentially infectious substances of human origin, in accordance with local regulations.


Ring reconditioning

For a proper, effective and safe reconditioning of the myRing Forte and their extremities read the Cleaning, Disinfection and Sterilization Instructions at www.polydentia.ch.

Tips & tricks using myQuickmat Forte starter kit


Tines removal (A) (B)

Tines must be disassembled before cleaning and sterilizing. To remove the tines correctly, it is important to preserve the internal anatomical wings. ✓ Pull from HERE. ✗ DO NOT pull from HERE.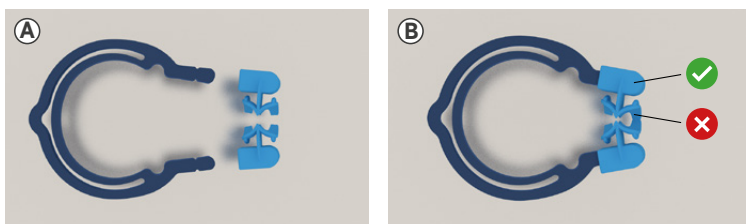


Tines placement on myRing Forte © ④

Tines are replaceable and supplied in pairs. You can easily detach the tines, avoid finger pressure on the internal anatomical wings to preserve them. Make sure tines are placed on myRingForte as shown in the picture. ⑤ CORRECT POSITIONING: The v-shaped section of myTines must face downwards to accommodate the wedge. CORRECT DISASSEMBLY: Remove myTines by pulling from their main body. Do not pull from their wings.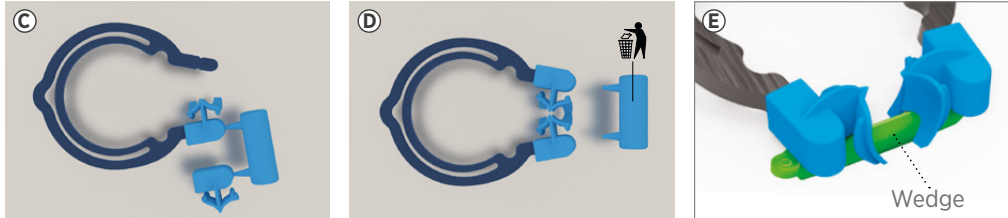


MOD and multiple tooth restorations ⑥

myQuickmat Forte rings are designed to be stacked over one another to perform a MOD and multiple tooth restorations. **1.** Prepare the cavities as described in the instructions for use, step 1. **2.** Place matrices and wedges on both mesial and distal surfaces of the tooth being restored. **3.** Position myRing Forte. **4.** Complete the restoration as described in the instructions for use, steps 5 to 8.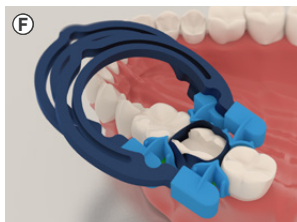


Restoration of adjacent cavities ⑥

myRing Forte tines are shaped to perfectly adapt the mesial and distal surfaces of two adjacent teeth. This feature allows the restoration of two adjacent cavities at the same time. **1.** Prepare the cavities as described in the instructions for use, step 1. **2.** Place matrices and wedges on both teeth being restored. **3.** Position myRing Forte. **4.** Complete the restoration as described in the instructions for use, steps 5 to 8.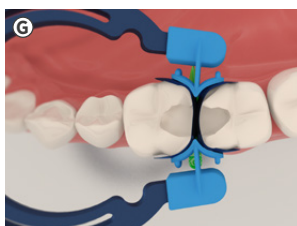

myRing Forte Reset ⑧ ①

If myRing Forte is over-expanded it can be retensioned by using Polydentia's myForceps as follows: **1.** Remove the tines first. **2.** Place myRing Forte within the notches on the forceps. **3.** Squeeze forceps closed to return myRing Forte in its perfect shape. ➡ Squeeze.

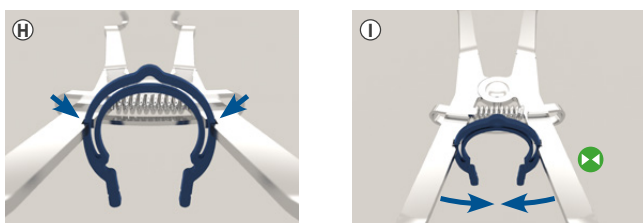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