Instructions for use

myQuickmat hoop kit

circumferential
matrix
system

en - ENGLISH	2	da - DANSK	32
de - DEUTSCH	5	cs - ČEŠTINA	35
fr - FRANÇAIS	8	tr - TÜRKÇE	38
it - ITALIANO	11	el - ΕΛΛΗΝΙΚΑ	41
es - ESPAÑOL	14	bg - БЪЛГАРСКИ	44
pt - PORTUGUÊS	17	hu - MAGYAR	47
no - NORSK	20	lv - LATVIEŠU VALODA	50
et - EESTI KEEL	23	lt - LIETUVIŲ KALBA	53
fi - SUOMI	26	ro - LIMBA ROMÂNĂ	56
sv - SVENSKA	29	sk - SLOVENČINA	59

Medical Devices Kit for professional use only
All individual components are CE marked
Circumferential Matrix System

Instructions for use

1. Product Description

myQuickmat Hoop kit circumferential matrix system is designed to be used as a matrix to contain direct-placement restorative materials in posterior cavities involving proximal surfaces. myQuickmat Hoop kit with integrated tightener/ retainer allows matrix placement and retention without the need for bulky retainers allowing for ease of placement, better access, clear view of the operating field and greater patient comfort. Access for wedge placement (where indicated) is not hindered by an external retainer. The single-use, disposable Quickmat Hoop circumferential preloaded matrices allow proper subgingival and proximal contact adaptation while protecting against cross-contamination.

1.1 Indication

myQuickmat Hoop kit is indicated where placement of a matrix around a prepared tooth will facilitate contouring of a direct placement restorative material (e.g., Class II preparations).

1.2 Contraindications

None Known.

1.3 Delivery Forms

Quickmat Hoop circumferential preloaded matrices are available in one thin gauge (thickness) in 3 heights: 5.5 mm, 6.5 mm and 8 mm to fit a wide range of tooth sizes and shapes.

1.4 Composition

Quickmat Hoop - circumferential preloaded matrices: Polished stainless steel and Polyacetal Thermoplastic.

myHoopy wedges - interdental plastic wedges: Polyacetal Thermoplastic.

myHooper - instrument: 316 stainless steel.

2. General Safety Notes

Be aware of the following general safety notes and the special safety notes in other chapters of these instructions for use.

Safety alert symbol

This is the safety alert symbol. It is used to alert you to potential personal injury hazards. Obey all safety messages that follow this symbol to avoid possible injury.

2.1 Warnings

1. myQuickmat Hoop kit consists of small, sharp parts. As with all intra-oral instruments, care must be exercised to ensure stable handling and placement. Placement of rubber dam or other suitable isolation is advised prior to operative procedures. Wear suitable protective eyewear and gloves.
2. Aspiration: If aspiration of matrices or wedges or any part occurs seek immediate medical attention.
3. Ingestion: If accidental swallowing occurs, encourage patient to seek medical assistance to locate swallowed part. If nausea or illness develop, seek medical attention immediately.
4. Laceration: matrices are sharp, and may cut unprotected skin.
5. Handling matrices and wedges with gloved hands is recommended. In the event a clean matrix causes a skin laceration, employ usual first aid measures. Discard and properly dispose of matrix and wedge.
6. Used matrices and wedges are contaminated. Follow proper exposure control plans. In the event of an accidental exposure, follow recommended post-exposure procedures. Following use of matrices and wedges, proper handling and disposal techniques are required. Used matrices are contaminated and sharp. When handling or disposing, use precautions as when handling or disposing other contaminated sharps.

2.2 Precautions

1. This product is intended to be used only as specifically outlined in the Instructions For Use. Any use of this product inconsistent with the Instructions For Use is at the discretion and sole responsibility of the practitioner.
2. Devices marked single use on the labeling are intended for single use only. Discard after use. Do not reuse in other patients in order to prevent cross-contamination.
3. Avoid placing excessive pressure in an apical direction when placing or on placed matrices. Apical force may result in injury to periodontal tissues. (See Warnings)
4. Only use the myHooper instrument to tighten the Quickmat Hoop circumferential preloaded matrices. (See Step-by-Step Instructions)

2.3 Adverse Reactions

1. Product misuse may cause failure resulting in respiratory, digestive, eye, skin, or mucosa injury. (See Contraindications, Warnings and Precautions)
 2. Failure to properly employ accepted handling and disposal techniques may present biohazard. (See Warnings and Precautions).
- Any serious incident in relation to the product should be reported to manufacturer and the competent authority according to local regulations.

2.4 Storage

Store myQuickmat Hoop kit at room temperature, away from moisture or excessive humidity.

3. Step-By-Step Instructions

3.1 Pre-Operative Sterilization

The Instrument must be sterilized before each use. Matrices and wedges are intended for single use, and are not supplied sterile. If desired, these components may be sterilized one time by vacuum steam autoclaving, 132°C, 4 minutes, prior to clinical use. Paper/plastic steam sterilization pouches may be used, but are not required. Follow vacuum steam autoclave manufacturer's Instructions for Use for loading and operation cycle.


3.2 Cavity Preparation

Complete cavity preparation as usual utilizing rubber dam or other suitable isolation technique.

Technique Tip: Pre-wedging to achieve slight separation may facilitate matrix placement through tight contacts, especially on the intact side of the tooth being restored.

3.3 Matrix Selection/Preparation-Enlarging Matrix Circumference

There are three heights of matrix band. Push the Instrument rotating shaft (A) forward in the instrument sleeve (B) to expand the instrument arms (C). Slide the instrument arms (C) into the V-shaped grooves (D) on the top and bottom of the plastic body of the retainer (E). This will cause the arms (C) to lock onto the plastic body of the retainer (E). If necessary, rotate the swivel (F) to the appropriate side for the working quadrant. To properly fit the tooth, matrix (G) should be slightly larger than the occlusal circumference. Twist the Instrument rotating shaft (A) counterclockwise to enlarge the loop, or clockwise to reduce the loop, if necessary.

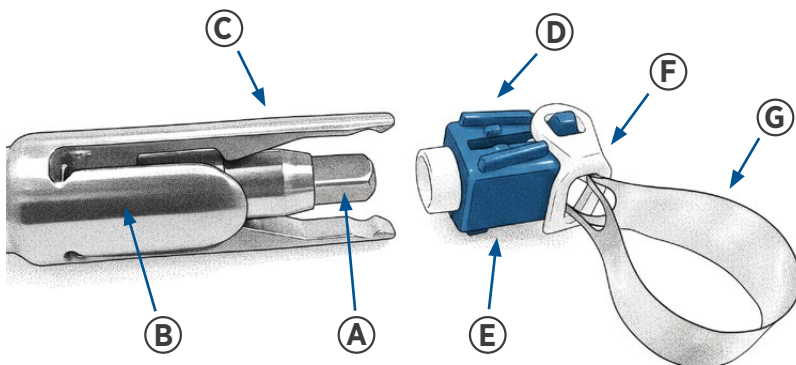3.4 Positioning Matrix

Carefully slide matrix around tooth, through the interproximal embrasures using a downward oscillating motion.

Technique Tips: To seat matrix in tight interproximal spaces – pre-wedging or a separator ring on posteriors may gain temporarily required space. To clear debris and facilitate seating – use waxed floss or a soft wedge between tight interproximal embrasures.

3.5 Tightening The Matrix

Stabilize the matrix with apical gentle finger pressure while tightening the matrix. Check to be sure gingival tissue is not trapped between matrix and tooth surface. If so, slightly lift and reposition matrix prior to tightening. Tighten the Instrument rotating shaft (A) by rotating it clockwise. When matrix is snug, and resistance is felt, STOP tightening. The matrix is properly subgingivally tightened. Push the Instrument rotating shaft (A) inwards to detach the Instrument from the Quickmat Hoop matrix.

3.6 Placement Of Restorative

Place wedges as desired (buccal and/or lingual). Burnish is usually unnecessary as the bands already have good proximal curvature. Place desired restorative material as per restorative material manufacturer's instructions for use.

3.7 Removing The Matrix

Following appropriate restorative curing/setting, remove any retaining rings and wedges, push the Instrument onto the Quickmat Hoop matrix and rotate the rotating shaft Ⓐ counterclockwise until matrix is loose. Bonding agents can cause the matrix to adhere to the composite so sliding a flat instrument between the matrix and the restoration can help to remove it. The matrix can also be cut in half with scissors. Remove the matrix taking care not to lacerate the gingivae.

Properly discard matrix.

Technique Tip: It is advisable to post-cure light cured restorative materials following matrix removal from all surfaces, including proximo-facial and lingual line angles.

Small parts-ingestion/inhalation

- Ensure the matrix is attached to the instrument arms during withdrawal from the mouth.
- Recover all pieces from the mouth immediately.

4. Hygiene

Cross-contamination – To reduce risk of Infection.

- Do not reuse single use products.
- Dispose of in accordance with local regulations.

Sterilization of matrices and wedges pre-use (if desired) and instrument (between each use) may be accomplished as outlined below:

Limitations on processing	Cold liquid immersion disinfection/sterilization, automated washer/disinfection have not been tested or validated for efficacy and are not recommended for use. Discard when corrosion and/or damage is noticed.
Packaging	Paper/plastic steam sterilization pouches may be used but are not required.
Pre-cleaning	Remove coarse impurities from the instrument as soon as possible after use with a soft brush and cold running water.
Sterilization	Pre-vacuum: Full cycle 132°C, 4 minutes. Follow manufacturer’s instructions for loading and operation cycle.
Drying	Use the drying cycle of the autoclave, minimum 30 minutes.
Storage	Ensure the sterilized package is completely dry before storing. Store at room temperature, away from moisture or excessive humidity and ensure sterilization pouch/container is not compromised. Instruments steam autoclaved unwrapped should be used immediately.

Cleaning, Disinfecting & Sterilizing

To disassemble the Instrument, apply firm pressure to push the instrument rotating shaft Ⓐ forward. This disassembles the Instrument, providing more thorough access for cleaning. Once you’ve finished cleaning, reassemble the Instrument by pressing the instrument rotating shaft Ⓐ firmly backwards until it audibly clicks into place.

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Rhondium Ltd