Instructions for use

PolydentiaPRO

matrix
systems

 polydentia
swiss manufacture 

en - ENGLISH	2
de - DEUTSCH	4
fr - FRANÇAIS	6
it - ITALIANO	8
es - ESPAÑOL	10
pt - PORTUGUÊS	12
no - NORSK	14
et - EESTI KEEL	16
fi - SUOMI	18
sv - SVENSKA	20

da - DANSK	22
cs - ČEŠTINA	24
tr - TÜRKÇE	26
el - ΕΛΛΗΝΙΚΑ	28
bg - БЪЛГАРСКИ	30
hu - MAGYAR	32
lv - LATVIEŠU VALODA	34
lt - LIETUVIŲ KALBA	36
ro - LIMBA ROMÂNĂ	38
sk - SLOVENČINA	40

Matrix Systems
Interproximal tooth protection

Instructions for use

Description

PolydentiaPRO matrix is a pre-shaped, contoured protection matrix intended to prevent damage to the adjacent tooth that commonly occurs during Class II preparations. The contoured shape and the two extremities are designed to easily slide under the contact point facilitating the placement of the matrix.

Composition

Stainless Steel.

Indications

Stainless steel matrix to be used during class II cavity preparation to prevent damage of the adjacent tooth.

Contraindications

None known.

Side effects

There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydentia.

Interactions with other medical devices or products

There are no known interactions.

Information for the patient

No information for the patient related to PolydentiaPRO is necessary.

Preservation and storage


Store in a clean, dry environment away from moisture or excessive humidity.

Warnings

Due to the presence of small and sharp components, the use of a rubber dam is mandatory to avoid accidental ingestion or aspiration. In case of accidental aspiration of PolydentiaPRO seek immediate medical attention. In case of ingestion of PolydentiaPRO, it is strongly recommended seeking for medical assistance and locate the ingested components. If illness develops after ingestion, consult a doctor immediately. Do not use as a sectional matrix.

Application

① Slide the protection matrix into the interproximal space. The concave side of PolydentiaPRO must be placed towards the tooth to be protected. In case of tight contact points, you can pull the protection matrix using tweezers. For a firm grip and easiest placement we suggest to use our myQuickmatrix Forceps, REF 5703. ② Secure PolydentiaPRO in place using an appropriately sized wedge. The wedge is placed between the tooth to be prepared and the protective matrix. The use of a wooden wedge allows the wedge to be customised during cavity preparation. ③ Proceed with the cavity preparation; in case of deep cavities you can customize the wood wedge according to the cavity shape. ④ Remove the wedge and PolydentiaPRO; if you customized the wedge, save it for the cavity restoration. ⑤ Place a sectional matrix of your choice and secure it with the wood wedge you customized before. Place a sectional matrix ring and proceed with the restoration of the cavity.

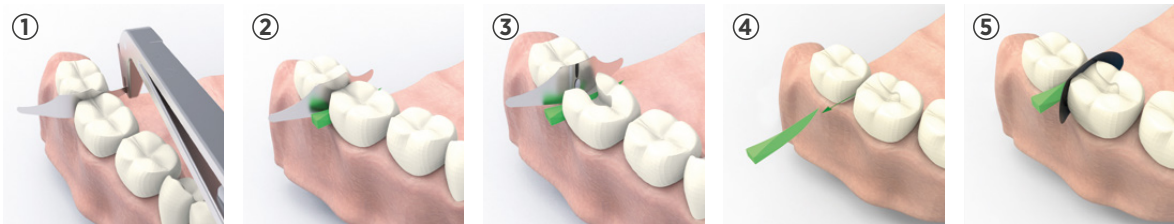Disposal

Because of their composition, matrices can not be sterilized and are therefore single use devices. Do not reuse them. Properly dispose devices contaminated with potentially infectious substances of human origin, in accordance with local regulations.

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