Instructions for use

# Quickmat deluxe

sectional  
matrix  
systems

|                       |    |                            |    |
|-----------------------|----|----------------------------|----|
| en - ENGLISH .....    | 2  | da - DANSK .....           | 32 |
| de - DEUTSCH .....    | 5  | cs - ČEŠTINA .....         | 35 |
| fr - FRANÇAIS .....   | 8  | tr - TÜRKÇE .....          | 38 |
| it - ITALIANO .....   | 11 | el - ΕΛΛΗΝΙΚΑ .....        | 41 |
| es - ESPAÑOL .....    | 14 | bg - БЪЛГАРСКИ .....       | 44 |
| pt - PORTUGUÊS .....  | 17 | hu - MAGYAR .....          | 47 |
| no - NORSK .....      | 20 | lv - LATVIEŠU VALODA ..... | 50 |
| et - EESTI KEEL ..... | 23 | lt - LIETUVIŲ KALBA .....  | 53 |
| fi - SUOMI .....      | 26 | ro - LIMBA ROMÂNĂ .....    | 56 |
| sv - SVENSKA .....    | 29 | sk - SLOVENČINA .....      | 59 |

Medical Devices Kit for professional use only  
All individual components are CE marked  
Sectional matrix systems

## Instructions for use

### Description

Quickmat Deluxe is a sectional matrix system leading to precise and anatomically correct contact points in the restoration of class II cavities. This kit features Polydentia's Quickring, a classic sectional matrix ring made of Stainless steel, and the insert "Re-Force" for Quickrings that can be added to the ring if there is a clinical need to increase the separation force in combination with the included Interdental Maple Wood Wedges. For the restoration of very extensive cavities, the kit also includes Silicone Tubes and Delta Tubes. These anatomically shaped silicone tubes ensure optimal matrix retention and adaptation to the tooth's anatomy. Quickmat sectional matrices complete the kit's assortment.

### Composition

Quickrings: stainless steel | Insert "Re-Force": thermoplastic | Quickmat matrices: stainless steel  
Silicone Tubes: silicone | Delta Tubes: silicone | Wood wedge: maple wood.

### Indications

Quickmat Deluxe Kit is indicated for gentle teeth separation in order to create correct contact points in the restoration of class II cavities.

### Contraindications

None Known.

### Side effects

There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydentia.

### Interactions with other medical devices or products

There are no known interactions.

### Information for the patient

No information for the patient related to Quickmat Deluxe is necessary.

### Shelf life

Please refer to the expiration date on the packaging. Expiration date refers to wooden wedges and silicone tubes.

### Preservation and storage

Store in a clean, dry environment away from moisture or excessive humidity.

### Warnings

Due to the presence of small and sharp components, the use of a rubber dam is mandatory to avoid accidental ingestion or aspiration. Always ensure the rings using dental floss. In case of accidental inspiration of a Quickmat Deluxe component (matrix, retaining ring and its extremities, wedge) seek immediate medical attention. In case of ingestion of a Quickmat Deluxe component, it is strongly recommended seeking for medical assistance and locate the ingested components. If illness develops after ingestion, consult a doctor immediately. All Polydentia reusable Products are supplied non-sterile and must be appropriately disinfected, cleaned and sterilized before the first use. Please refer to the Cleaning, disinfection and Sterilization Instructions at [www.polydentia.ch](http://www.polydentia.ch).

## Application

① **Prepare the cavity:** Remove the carious lesion and prepare the tooth with a conservative Class II preparation. Even if the contact was not totally broken the matrix can be placed. ② **Place the matrix:** Select an appropriate matrix band according to the tooth morphology and restoration type. Place the contoured sectional matrix with the concave edge towards the occlusal margin using your fingers or with tweezers. ③ **Wedging:** Select the appropriate sized wooden wedge to close the gingival margin, and insert it firmly. When needed maintain a finger on the matrix band to avoid dislodging during wedging. ④ **Place Quickring:** Expand Quickring using Polydentia's myForceps or standard rubber dam forceps. Place the ring carefully and securely. Please refer to the Tips & Tricks section to see the different ways to position Quickring and ensure best teeth separation as well as matrix adaptation. Add the "Re-Force" insert in case you need to increase the separation force of the ring. **Burnish the matrix:** For an excellent contact burnish the band against the adjacent tooth, using a ball burnisher or tweezer. ⑤ **Restore the cavity:** Restore the cavity according to the filling material manufacturer's instructions. **Remove the sectional matrix system:** Remove Quickring, the wedge and the matrix. Our suggestion for an easy matrix removal is to use the special Polydentia myQuickmatrix Forceps. **Finishing and polishing:** With thin burs remove any small excess of material then polish the restoration according to manufacturer's recommendations.

## Disposal

Because of their composition, matrices and wedges can not be sterilized and are therefore single use devices. Do not reuse them. Properly dispose devices contaminated with potentially infectious substances of human origin, in accordance with local regulations.

## Ring reconditioning

For a proper, effective and safe reconditioning of the Quickring and their extremities read the Cleaning, Disinfection and Sterilization Instructions at [www.polydentia.ch](http://www.polydentia.ch).

## Tips & tricks using Quickmat Deluxe

### Wedging

According to the clinical situation, the wooden wedge can be broken in order to better accommodate Quickring with Silicone Tubes. This trick is very helpful in case of small teeth.

### Silicone Tubes placement options

Cylindrical silicone tubes can be placed on Quickrings tines for use on large cavities. Ⓐ Delta Tubes can be oriented according to the clinical situation to obtain the best matrix retention and adaptation to the tooth's anatomy. Different orientation possibilities can be combined to best adapt Quickring to each specific clinical situation.

## Case small cavities

Delta Tubes edges allow a perfect adaptation between the teeth.

## Case broad cavities


Delta Tubes flat surfaces provide support on the adjacent tooth, no matrix deformation.

## Quickring without Silicone Tubes or Delta Tubes

The use of Silicone Tubes or Delta Tubes is not mandatory, Quickring can be placed also without ring extremities Quickring tines placed near to the wood wedge. This placement technique is suitable for cases of a wide space in the interproximal area. Quickring tines placed over the wood wedge. This placement technique is suitable if the interproximal space is tight or when the cavity is deep.

## MOD and multiple tooth restorations <sup>®</sup> <sup>©</sup>

Quickrings are designed to be stacked over one another to perform a MOD and multiple tooth restorations. **1.** Prepare the cavities as described in the instructions for use, step 1. **2.** Place matrices and wedges on both mesial and distal surfaces of the tooth being restored. **3.** Position Quickring. **4.** Complete your restoration as described in the instructions for use, steps 5 to 8.

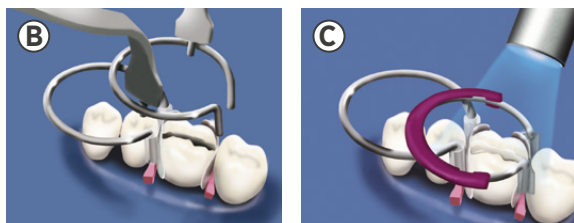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