Instructions for use

unica introkit

in cooperation with **STYLE ITALIANO™**

**matrix
systems**

 **polydentia**
swiss manufacture 

en - ENGLISH	2
de - DEUTSCH	4
fr - FRANÇAIS	6
it - ITALIANO	8
es - ESPAÑOL	10
pt - PORTUGUÊS	12
no - NORSK	14
et - EESTI KEEL	16
fi - SUOMI	18
sv - SVENSKA	20

da - DANSK	22
cs - ČEŠTINA	24
tr - TÜRKÇE	26
el - ΕΛΛΗΝΙΚΑ	28
bg - БЪЛГАРСКИ	30
hu - MAGYAR	32
lv - LATVIEŠU VALODA	34
lt - LIETUVIŲ KALBA	36
ro - LIMBA ROMÂNĂ	38
sk - SLOVENČINA	40

Medical Devices Kit for professional use only
All individual components are CE marked
Matrix systems

Instructions for use

Description

Unica anterior and Unica minideep anterior, thanks to their contoured shape, adapt correctly to the different morphologies of anterior teeth and are intended to restore interproximal and cervical margins at once, even in presence of rubber-dam or gingival retraction cords reducing chair-time significantly. Furthermore, Unica anterior and Unica minideep anterior matrices, once positioned, allow to easily visualize the final shape of the restoration. Unica anterior & Unica minideep anterior allow to: restore both proximal and cervical margins at once, anatomically restore the proximal margins thanks to its contoured profile, easily manage the cervical area.

Composition

Unica anterior & Unica minideep anterior: stainless steel | myWedge: thermoplastic | myQuickmatrix Forceps: stainless steel.

Indications

Unica anterior introkit is indicated for: class III cavity restorations, class IV cavity restorations, class V cavity restorations, direct stratification composite veneers, shape modifications.

Contraindications

None known.

Side effects

There are no known side effects. Promptly report any serious incident by contacting your distributor or Polydentia.

Interactions with other medical devices or products

There are no known interactions.

Information for the patient

No information for the patient related to Unica anterior Introkit is necessary.

Preservation and storage

Store in a clean, dry environment away from moisture or excessive humidity.

Warnings


Due to the presence of small and sharp components, the use of a rubber dam is mandatory to avoid accidental ingestion or aspiration. In case of accidental inspiration of a Unica anterior Introkit component seek immediate medical attention. In case of ingestion of a Unica anterior Introkit component, it is strongly recommended seeking for medical assistance and locate the ingested components. If illness develops after ingestion, consult a doctor immediately. All Polydentia reusable Products are supplied non-sterile and must be appropriately disinfected, cleaned and sterilized before the first use. Please refer to the Cleaning, disinfection and Sterilization Instructions at www.polydentia.ch.

Disposal


Because of their composition, matrices and wedges can not be sterilized and are therefore single use devices. Do not reuse them. Properly dispose devices contaminated with potentially infectious substances of human origin, in accordance with local regulations.

Application

Select Unica matrices: Choose Unica according to its shape and the tooth to be restored. ① Select Unica anterior for: maxillary central incisors, maxillary lateral incisors, canine teeth. ② Select Unica minideep anterior for: maxillary and mandibular laterals, mandibular central incisors, conoid teeth, triangular teeth, peg laterals, teeth with narrow cervical diameter.

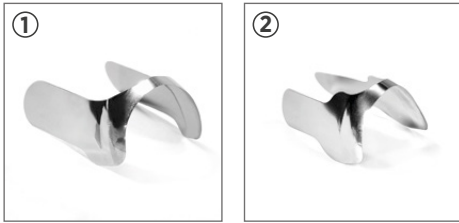
How to position Unica matrices: ③ Use the placement wings to insert Unica proximally. In case of tight contact surfaces, insert one wing at a time. ④ Gently pull the matrix palatally by its placement wings. ⑤ Push the cervical connection into the gingival sulcus. ⑥ Insert myWedge to fix the matrix in position.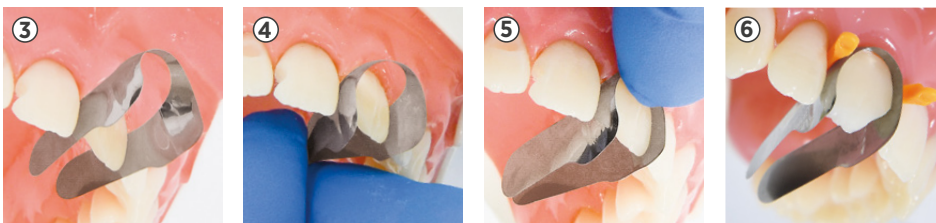

Tips and tricks

Secure Unica matrices in place with a liquid dam, in case a wedge risks displacing or deforming the matrix. ✓ Yes. ✗ No.

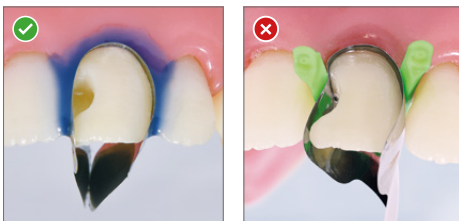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